



XtandR

Important!

Carefully read before first use

USAGE

- The bite plates are only intended for taking X-rays of the incisors and have to be taken out of the horse's mouth immediately after taking the X-rays.
- Never leave the horse unattended with the bite plates in its mouth.
- The biteplates have been tested according to the official DAkkS-DKD-R 3-3 regulation and can withstand a pressure of 5,13 KN/ 513 Kg
We will send you the test results on request
- As for the usage of these bite plates and our products in general. we further ask your attention for our **General Terms and Conditions** and especially paragraph 8.6 thereof which states:

Podoblock B. V. shall never be liable for any nonconformities of the products or any damage caused by the use of the products if the products have been used in a manner that differs from that stated in the relevant instructions for use or have been put under a higher pressure of higher weight than the maximum weight or pressure stated in the instructions for use.

If requested to do so by Podoblock B. V., counterparty shall prove that the products have been used in accordance with the relevant instructions for use and that the nonconformity or damage has not been caused by hitting with the legs, rearing, jumping (away) or another movement or reaction by the relevant animal, not controlled by the assistant.



www.podoblock.com